



www.microchipIDequine.com

John R. Wade, DVM

Microchip ID Equine, Inc
(800) 434-2843

Fax: (985) 796-1531

81447 Highway 25 ♦ Folsom, Louisiana 70437

Note: Always scan the horse prior to injecting. If a microchip is found, call (800) 434-2843 for information.



1. Locate the injection site. On the left side, halfway between the poll and withers, locate the nuchal ligament, just below the mane.



2. Prepare the site for aseptic injection.

3. Standard Injection: Inject the microchip directly into the nuchal ligament, perpendicular to the neck. (On an angle for younger horses.) Slowly depress the syringe plunger until it stops. Apply pressure on the injection site as the needle is withdrawn.



4. Scan the horse.



5. Record the microchip number.



6. Enroll in *Equine Protection Registry*™

